

Quick Guide

How to Scroll Through/Browse

1. Scroll Middle Mouse Button
2. Click to Activate Browse Button on The Menu Bar. Click & Drag Left Mouse Button




How to Select Presets

1. Type Numbers from 0 - 5 on the keyboard

0 = Default 3 = Lung
 1 = Bone 4 = Brain
 2 = Organs 5 = Angio

How to measure HU

- Click on Drop Down Arrow on the Measuring tools and choose ROI tool and Draw circle on the area

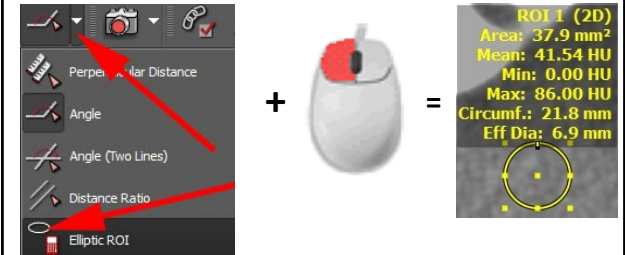

How to Change Window Level

1. Click Both Left & Right Button & Drag Mouse
2. Click to Activate Window Level Button on The Menu Bar. Click & Drag Left Mouse



How to Select MPR

- Click on MPR tab on Tool Bar area



How to Remove Patient Name & Line

If you need to remove Patient Name and Crosshair Line, press on Space bar once.

Press on Space bar again, to display Patient Name/ Lines back.




How to Zoom/Pan

1. Click /Hold down Middle Button & Drag Mouse to Zoom
2. Click on Drop Down Arrow & select Zoom or Pan Button to Activate. Click & Drag Left Mouse



How to Use Measuring Tools

- Click on Drop Down Arrow on Measuring Tools, choose the tool

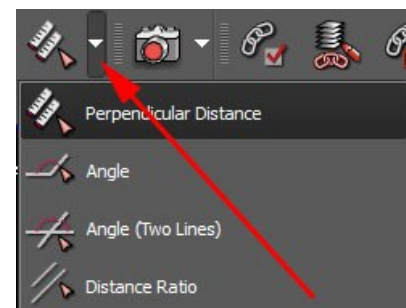

How to Add/Find More Tools

- If you can not see The tool that you need, to take a following steps:

1. Right Click on the Image Area, to select for more tools.



2. Click on Down Arrow on any Tool Icon to expand more tools

